

PRIME|ONE MICRO APPLICATORS

Disposable micro applicators



CONFIGURATION OVERVIEW:

- Material:** Medical level Polypropylene handle, Nylon Brush
- Type:** Non-Sterile
- Description:**
- Ideal for application of sealants, bonding agents, etchants, hemostatic solutions, cavity liners and disclosing solutions.
 - Non-absorbent, non-linting fibers, suspend solutions as small as 1/8 of a drop without spilling, dripping or waste.
 - The bendable neck stays in place to facilitate the placement of material in difficult areas.
 - Flexible plastic handle to pinpoint accuracy application of any liquids.

Class 1 Medical Device according to EU MDR 20017/745 according to rule 5 Annex VIII

CONFORM TO : EN ISO 14971:2019/A11:2021, EN ISO 15223-1:2021/Amd.1:2025, EN ISO 10993-1:2021, EN ISO 10993-5:2009, EN ISO 10993-10:2010, EN ISO 20417:2021

TECHNICAL SPECIFICATIONS:

Size	Regular	Fine	Superfine
Length of the applicator (including the fiber tip) (mm) ± 0.1 mm	102	101.5	100.2
Diameter of the fiber tip (mm) ± 0.3 mm	3	1.9	1.5
Length of the fiber tip (mm) ± 0.5 mm	3.5	3.5	2.5

SHELF LIFE : 5 years

PRODUCT VARIANTS:

Regular	Art-No.	MICAP25-AKZ-BL-REG	MICAP25-AKZ-LI-REG	MICAP25-AKZ-FG-REG	MICAP25-AKZ-PI-REG
	Color	Med Blue	Lilac	Fresh Green	Pink
					

Fine	Art-No.	MICAP25-AKZ.YE-FNE	MICAP25-AKZ-MI-FNE	MICAP25-AKZ-MC-FNE	MICAP25-AKZ-LU-FNE *	MICAP25-AKZ-CE-FNE *
	Color	Yellow	Mint Green	Magic Blue	Laguna Blue	Crispy cedro
						

Superfine	Art-No.	MICAP25-AKZ-WH-SFN
	Color	White
		

* Colours available upon request



Akzenta International SA
Via Giuseppe Motta, 24
CH-6830 Chiasso

tel. +41 (0) 91 921 14 92
fax. +41 (0) 91 682 85 20

www.akzenta.com
e-mail: business@akzenta.com

PACKAGING:

Pcs/dispenser	100
Dispenser/box	4
Boxes/carton	40
Carton size	52x42x24.5 cm
Carton weight	16 Kg

STORAGE AND HANDLING:

In dry, dark, dust-free and ventilated space at room temperature. Keep away from direct sunlight.

INSTRUCTIONS FOR USE:

Choose the suitable size of applicators to be coated with medicine to be applied to the affected area.

WARNING:

- The fiber brush cannot be used for a long time with absorbed large volume of chemical solution. Avoid long usage to prevent the fiber brush to fall out.
- Do not use with reagents containing BUTANONE.



Rev. 0 11-08-2025

