FD 322 top wipes

- DE** Schnelldesinfektion
Gefahr
- EN** Quick-acting disinfection
Danger
- FR** Désinfection rapide
Danger
- IT** Disinfezione ad azione rapida
Pericolo
- ES** Desinfección de acción rápida
Peligro
- PT** Desinfecção de acção rápida
Perigo
- NL** Snel-desinfectie
Gevaar
- SV** Snabbdesinfektion
Fara
- FI** Nopeasti tehoava
desinfiointiaine
Vaara
- NO** Hurtigvirkende desinfeksjon
Fare
- DA** Hurtigvirkende desinfektion
Fare
- EL** Ταχεία απολύμανση
Κίνδυνος
- PL** Szybka dezynfekcja
Niebezpieczeństwo
- CS** Dezinfekce s rychlým účinkem
Nebezpečí
- SK** Rýchlo účinná dezinfekcia
Nebezpečenstvo
- SL** Hitro učinkovito razkužilno
sredstvo
Nevarno
- HR** Brza dezinfekcija
Opasnost
- HU** Gyors fertőtlenítős
Veszély
- EE** Kiirdesinfitseerimine
Ettevaatust
- LV** Ātrā dezinfekcija
Bīstami
- LT** Greitas dezinfekavimo poveikis
Pavojinga
- UA** Швидка дезінфекція
Небезпечно
- BG** Бързодействащ дезинфектант
Опасно
- RO** Dezinfecția rapidă
Pericol
- TR** Hızlı Dezenfeksiyon
Tehlike

1-Propanol
CAS-No. 71-23-8

 rochemie GmbH + Co. KG
Max-Planck-Straße 27
70806 Kornwestheim, DE
a DÜRR DENTAL company
Tel. +49 7142 705-0
info@duerrdental.com
www.duerrdental.com
Made in Germany

 **DÜRR
DENTAL**

FD 322 top wipes

Schnelldesinfektion

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☎ +41 448850634

EN **FD 322 top wipes** Quick-acting disinfection

FD 322 top wipes are aldehyde-free, alcoholcontaining ready-to-use disinfecting wipes for rapid disinfection of alcohol-resistant surfaces of invasive and non-invasive medical devices and medical furnishings in hospitals, private practices, and laboratories. FD 322 top wipes are distinguished by fast efficiency and excellent material compatibility. Disinfected surfaces dry rapidly.

Composition: FD 322 top wipes contain FD 322. 100 g active substance solution contain 32 g 1-propanol, 26 g ethanol together with additives and water.

Spectrum of activity: Bactericidal, tuberculocidal, fungicidal, with virucidal activity against enveloped viruses (enveloped viruses incl. HBV, HCV, HIV, coronaviruses) and limited spectrum of virucidal activity (incl. rotaviruses, adenoviruses, noroviruses), polyomavirus SV40. FD 322 is included in the **VAH and IHO lists of disinfectants**. The active substance solution has been tested under high load in accordance with EN 13727, EN 13624, EN 14348, EN 14476, EN 16615, as well as DVV/RKI guidelines. FD 322 top wipes are tested as ready-to-use wipes in 4-field test in accordance with VAH/EN 16615.

Application: Remove FD 322 top wipes individually. Wipe the surfaces and objects to be disinfected and allow the product to take effect. Disinfect visibly contaminated surfaces with FD 322 top wipes and remove contamination. Disinfect cleaned surfaces again. Make sure that wetting is complete throughout the entire exposure time. Close the flow pack immediately following use. In accordance with VAH, the action time is 1 minute (bactericidal / yeasticidal), 30 seconds to 1 minute for Tb bacteria, and 10 to 15 minutes for mould. Inactivation of viruses takes place within 30 seconds to 5 minutes (see table). For further information, see product information on the Internet.

Note: Disinfect the exterior surfaces of medical devices, such as intraoral cameras, nozzles, handpieces, and contra-angle handpieces prior to processing as required in accordance with the manufacturers' instructions. Use caution with alcohol-sensitive materials (e.g. acrylic glass or similar). In the case of sensitive surfaces, test for compatibility in an inconspicuous place. Changes in colour and odour of FD 322 top wipes during storage have no impact on efficacy. Use exclusively by qualified personnel. All serious events that occur in association with the product must be immediately reported to the manufacturer and the competent authority of your member state.

  see imprint.

Danger. Flammable solid. Causes serious eye irritation. May cause drowsiness or dizziness. Keep away from heat, hot surfaces, sparks, open flames and other ignition sources. No smoking. Wear

protective gloves. IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. Store in a well-ventilated place. Keep container tightly closed. Dispose of contents/container to hazardous or special waste collection point.

UK-REP: DÜRR DENTAL (Products) UK Ltd.
14 Linnell Way – Telford Way Industrial Estate
Kettering, Northants NN16 8PS, United Kingdom
☎ +44 1536526740

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