FD multi wipes

FD multi wipes compact

| | | | |
|----|--------------------------------|----|--------------------------------|
| DE | Vliestücher | CS | Netkaných ubrousků |
| EN | Non-woven wipes | SK | Utierka z rúna |
| FR | Lavettes non-tissée | SL | Netkani krpe |
| IT | Panni in tessuto non-tessuto | HR | Maramice od filca |
| ES | Toallitas de vellón | HU | Vlieskendő |
| PT | Toalhetes em tecido não tecido | EE | Fliislapid |
| NL | Vliesdoekjes | LV | Neaustas salvetes |
| SV | Non-woven-servetter | LT | Neaustinio pluošto servetėlės |
| FI | Kuituliinoja | UA | Серветки з нетканого матеріалу |
| NO | Fleecekluter | BG | Нетъкани кърпи |
| DA | Non woven-klude | RO | Lavete nețesute |
| EL | Πανάκια από μη υφασμένο υλικό | TR | Vlies bez |
| PL | Ściereczki włókninowe | | |

 rochemie GmbH + Co. KG
Max-Planck-Straße 27
70806 Kornwestheim, DE
a DÜRR DENTAL company
Tel. +49 7142 705-0
info@duerrdental.com
www.duerrdental.com
Made in Germany

 DÜRR
DENTAL

Z.358-2020-11

DE

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☎ +41 448850634

EN

FD multi wipes/FD multi wipes compact

Non-woven wipes

FD multi wipes / FD multi wipes compact for disinfection of surfaces are suitable for soaking, by the user, with liquid Dürr system hygienic surface disinfectants. Please see the comprehensive product information on the Internet for for more information about dosing and use as well as for further details. The safety notices are included on the respective product label.

Recommended procedure:

- 1) Follow instructions for use.
- 2) Wear gloves.

- 3) Before refilling or replacing the product, remove remaining non-woven wipes and liquid from the dispenser box and also dispose of the label from the lid. Manual processing of the dispenser box and lid: Mechanically scrub and remove any deposits or residues. Thoroughly rinse off dispenser box including lid inside and out with hot tap water ($> 55^{\circ}\text{C}$ – NB: danger of scalding!) and dry with disposable wipe. Then, using a disposable wipe impregnated with alcoholic fast-acting disinfectant (e.g. FD 322 or FD 333), completely wet and thoroughly disinfect dispenser box and lid (allow sufficient time of action!). Make sure to dry the dispenser and lid completely. Alternative automated processing: Process and dry the dispenser and lid in a chemothermal cleaning and disinfection system at up to 93°C . Do not process contaminated dispenser boxes, dispose of them. Insert roll of wipes into the cleaned dispenser box.
- 4) Uniformly impregnate non-woven FD multi wipes with 2.5 l solution of active substance, or FD multi wipes compact with 1.4 l active substance solution.
- 5) Thread in first wipe, close lid and wait for the appropriate time (30 to 60 minutes) before using the first wipe.
- 6) Paste newly written label on the lid. Remove a wipe and wipe the surface to be disinfected completely. Ensure time of action in accordance with the directions for use of the chosen disinfectant. Close dispenser box tightly after each removal of a wipe. After impregnation, the roll of wipes can be used for 28 days or 3 months (FD 322, FD 333, FD 333 forte, FD 366 sensitive).

Freely protruding wipes may be contaminated and must be discarded. Do not push removed wipes back into the dispenser box. Do not use wipes that have dried out. Replace defective dispenser boxes.

Note: For professional use only. All serious events that occur in association with the product must be immediately reported to the manufacturer and the competent authority of your member state.

DÜRR DENTAL (Products) UK Ltd.
14 Linnell Way – Telford Way Industrial Estate
Kettering, Northants NN16 8PS, United Kingdom
☎ +44 1536526740

FR

FD multi wipes/FD multi wipes compact

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Molenheidebaan 97, 3191 Hever, Belgium

☎ +32 15616271

IT **FD multi wipes/FD multi wipes compact**

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